


NEW USER

Initial eVac System

Bio-Pure Restore Treatments

Per 2 Operatories

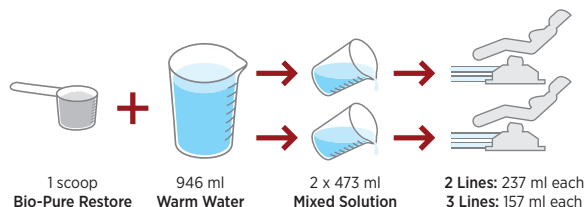

Recommended Product

2801101

Bio-Pure Restore Kit *Order 1 kit per 4 operatories*

Includes 6 oz. Bio-Pure Restore, SE-HVE dispenser, \$25 rebate coupon (to be used towards next purchase of 3 x 14 oz. or 1 x 48 oz.)

Instructions (per 2 operatories)

- › Begin the restore protocol at end of business on the last day of the work week
- › Add 1 scoop (5 g) Bio-Pure Restore to 32 oz./946 ml of warm water, mix thoroughly
- › If operatory has 2 lines, aspirate/vacuum 8 oz./237 ml per line
- › If operatory has 3 lines, aspirate/vacuum 5-1/3 oz./157 ml per line
- › Inspect traps twice per day and clean as necessary

Duration of treatment, based on age of practice

1-4 Years

Repeat daily for 5 business days

4+ Years

Repeat daily for 10 business days

15+ Years/Slow Suction

Restore **only 2 operatories** at a time


- › Begin protocol with the two operatories nearest the pump
- › Follow the above instructions daily for 10 business days or once Biofilm/plaque release ends at the chairside traps
- › After first 2 operatories are clear, and after 10 business days proceed with the next 2 operatories, working away from pump
- › For optimal results, involve a technician during initial restore introduction—for example, technician visits following Monday and drops off backup amalgam filter

EXISTING USER

Ongoing eVac System

Maintenance Treatments

Twice Weekly Maintenance Protocol
Per Operatory

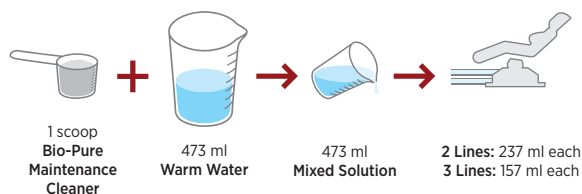

Recommended Products

2801001

6 oz. Bio-Pure Maintenance Cleaner (4 operatories, 1 month supply)

2801002

14 oz. Bio-Pure Maintenance Cleaner (4 operatories, 3 months supply)

2801003

48 oz. Bio-Pure Maintenance Cleaner (4 operatories, 9 months supply)

OPTION 1

Twice Weekly Maintenance Instructions (per operatory)

- › At closing add 1 scoop (5 g) Bio-Pure to 16 oz./473 ml of warm water, mix thoroughly
- › Aspirate/vacuum into operatory lines dividing solution evenly between each line (8 oz./236 ml for 2 lines, 5-1/3 oz./157 ml for 3 lines); proceed one line at a time
- › Procedure requires 2 treatments per week at end of day

OPTION 2

Daily Maintenance Instructions (per 2 operatories)

- › At closing add 1 scoop (5 g) Bio-Pure to 32 oz./946 ml of warm water, mix thoroughly
- › Aspirate/vacuum into 1 operatory line, dividing 16 oz./473 ml solution evenly between each line (8 oz./237 ml for 2 lines, 5-1/3 oz./157 ml for 3 lines); proceed one line at a time
- › Treat each operatory daily at the end of the work day

Daily Maintenance Protocol
Per 2 Operatories



Helpful Hints

- › Any extra Bio-Pure Restore product may be used for the maintenance protocol
- › You may continue your regular chemical cleaning protocol for operatories not using Bio-Pure
- › Never place a wet scoop into Bio-Pure container
- › Re-seal lid to ensure freshness
- › Store out of direct sunlight and not in refrigerator
- › Rinse dispenser and pour residual mixture down sink to put microbes to work there
- › After system restore, pump settings may need to be reset to original OEM specifications
- › Also, each month mix 1 scoop (5 g) of Bio-Pure with 24 oz./710 ml warm water and pour down cuspidor or other drains to keep them clean

bowen